

Assembly of the CR 7 VET Replacement input section set



Repair work on the Scanner beyond normal maintenance procedures must only be carried out by DÜRR MEDICAL trained and certified professionals or our customer service.

The assembly and user instructions manual for the device must also be followed. This gives important information on safety, set-up, electrical connections, disinfection, cleaning, etc.

Delivery contents


Input section set2137-995-50

1x Retaining plate incl. guard lips


1x Cover incl. guard lips

Tools required

None

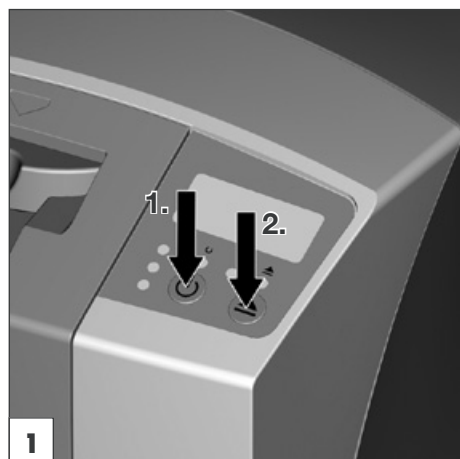


- Switch on the Scanner.
- Press the cleaning activation button.

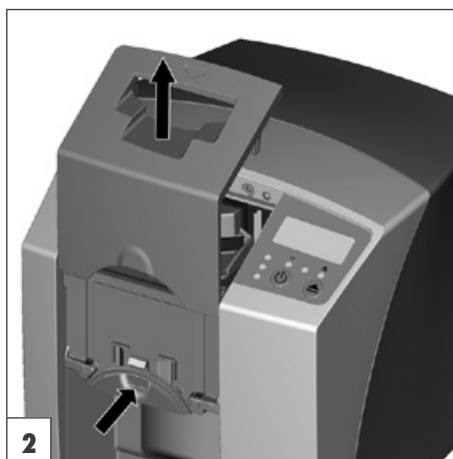


- Press the unlocking button while at the same time lifting the cover up.




- Take out the retaining plate.

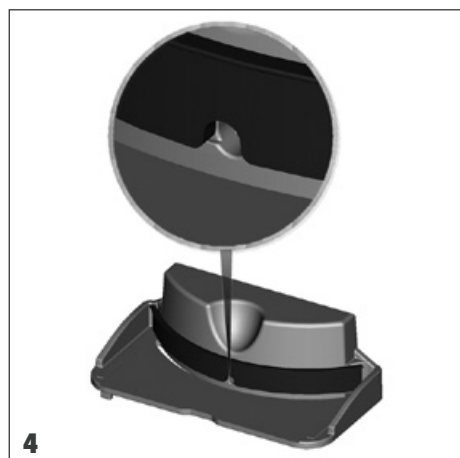


- The guard strip must not obstruct the hole for the light barrier.

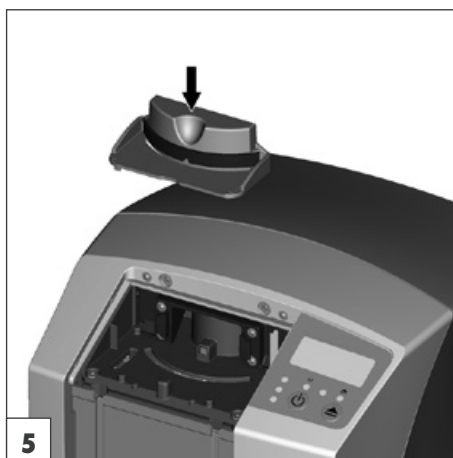

- Insert the retaining plate.

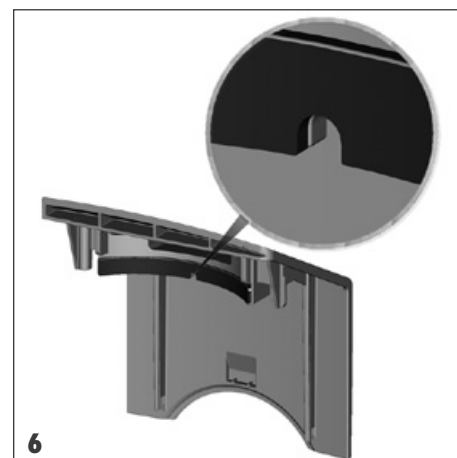


- The guard strip must not obstruct the hole for the light barrier.

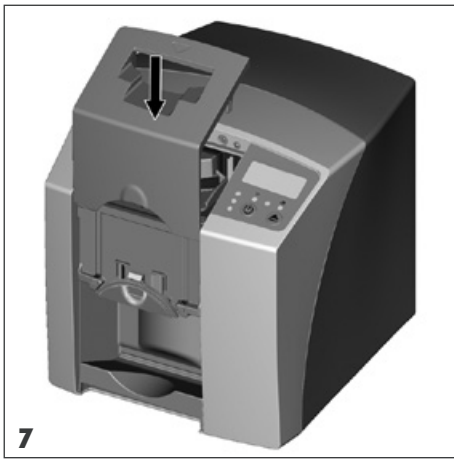

7

- Place cover in position.

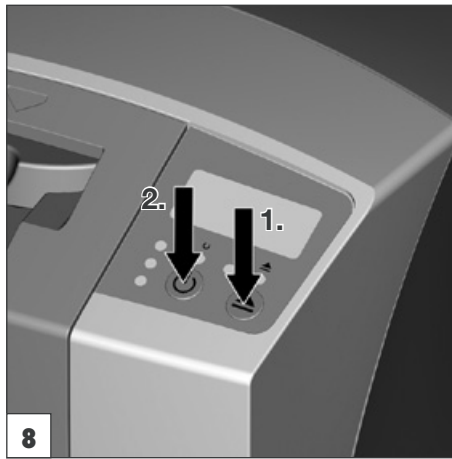

8

- Press the cleaning activation button.
- Switch off the Scanner.

Test 1:

- Switch unit on.
- Monitor the initialisation sequence (text is displayed on panel clearly, locking plate moves, drive starts, erasure unit lit).



There must be no unusual noises and nothing should appear on the display which are not expected from the unit.

Test 2:

- Scan an evenly homogenously exposed image plate.
- Check the X-ray for signs of artefacts or irregularities.



There must be no signs of artefacts or irregularities.

