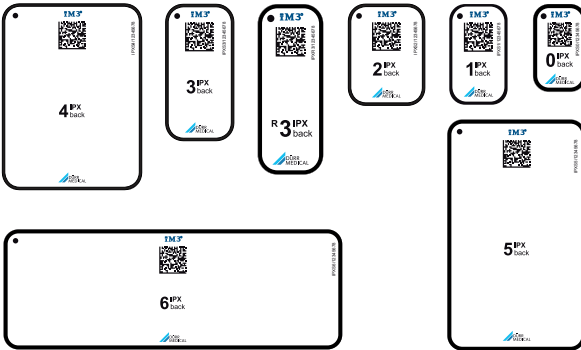
Speicherfolien IPX

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IPX image plates

The following information supplements the installation and operating instructions of the unit.

As a general rule, the installation and operating instructions for the device must also be observed. These instructions contain important information including safety instructions and information on setup, electrical connections, commissioning, the disinfection process, the cleaning process, etc.

Description	Size (cm)	Dürr Medical order number	iM3 order number
Size 0	2.2x3.5	2134104050	IPXS0
Size 1	2.4x4.0	2134104150	IPXS1
Size 2	3.1x4.1	2134104250	IPXS2
Size 3	2.7x5.4	2134104350	IPXS3
Size 4	5.7x7.6	2134104450	IPXS4
Size 5	5.7x9.4	2134104550	IPXS5
Size 6	5.1x14.0	2134104650	IPXS6
Size R3	2.2x5.4	2134104750	IPXR3

1 Intended Use

The image plates are not suitable for human medical applications.

The reusable image plates are designed for storing intraoral and extraoral (extremities) X-ray images in veterinary medical applications.

The image plates must only be used in combination with DÜRR MEDICAL light protection covers.

2 Usage

2.1 Correct use of image plates

 IPX image plates can only be read with a DÜRR MEDICAL image plate scanner that is equipped with an RFID reader (e.g. CR 8 VET).

WARNING

Risk of cross contamination when not using the light protection cover or when using the light protection cover more than once

- › Do not use an image plate without a light protection cover.
- › Do not use the light protection cover more than once (disposable item).

CAUTION

The image data on the image plate is not permanent.

The image data is altered by light, natural X-ray radiation and scattered X-ray radiation. This will lead to a reduction in diagnostic information and clarity.

- › Read the image data within 30 minutes of exposure.
- › Never handle exposed image plates without the light protection cover.
- › Do not subject an exposed image plate to X-ray radiation before or after the scanning process.
- › Do not X-ray during the scanning process if the unit is in the same room as the X-ray tube.

CAUTION

Image plates are toxic

Image plates that are not packed in a light protection cover can lead to poisoning when placed in the mouth of the animal or if swallowed.

- › Only place image plates in the mouth of the animal in a light protection cover.
- › The image plate or parts of it must not be swallowed.
- › If the image plate or parts of it have been swallowed, take appropriate measures and remove the image plate.
- › If the light protection cover has been damaged in the mouth of the animal, rinse the mouth thoroughly with lots of water. Do allow the water to be swallowed in the process.

- › Image plates must only be read using an image plate scanner that is approved by DÜRR MEDICAL.
- › Image plates are flexible like X-ray film. However, the image plates should not be bent.
- › Do not scratch the image plates. Do not subject the image plates to pressure from hard or pointed objects.
- › Do not soil the image plates.
- › Protect the image plates against sunlight and ultraviolet light.
Store image plates in a light protection cover or intraoral/extraoral foil cassette of the correct size.
- › Image plates will be pre-exposed on exposure to natural radiation and stray x-ray radiation. Protect erased and exposed image plates from X-ray interference.
If the image plate has been stored for longer than one week, erase the image plate prior to use.
- › Do not store image plates under hot or moist conditions. Note the ambient conditions.
- › When used properly, image plates can be exposed, read and erased several hundred times provided there is no mechanical damage.
Replace the image plate if there are any signs of damage, e.g. if the protective layer is damaged or there are visible scratches that impair the quality of the diagnosis.
Also replace the image plate if the RFID tag is damaged or becoming detached.
- › Image plates that have a production or packaging defect will be replaced by DÜRR MEDICAL in the same quantity. Claims can only be accepted within 7 working days after receipt of the goods.
- › Clean the image plates thoroughly.

2.2 Cleaning

Cleaning and disinfection wipes are unsuitable for cleaning image plates and may cause damage to them. Only use a cleaning agent that is compatible with the materials:

DÜRR MEDICAL recommends the Image Plate Cleaning Wipe (order number: CCB351A0101). Only this product has been subjected to material compatibility testing by DÜRR MEDICAL.

NOTICE

Heat or humidity will damage the image plate.

- › Do not steam sterilise the image plate.
 - › Do not immersion-disinfect the image plate.
 - › Only use cleaning agents that are compatible with the materials.
- › Soiling on both sides of the image plate should be cleaned off with a soft, lint-free wipe prior to every use.
 - › Remove resistant or dried on dirt with the image plate cleaning wipe. When doing this, follow the instructions for use for the cleaning wipe.
 - › Allow the image plate to completely dry before using it.

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Europe: iM3 Dental Limited
USA: iM3 Inc.
Australia/Corporate: iM3 Pty Ltd
DÜRR NDT GmbH & Co. KG