



Replacing the wiper blade set - CR 7 Plus

The following information is supplementary to the appliance Installation and Operating Instructions.

Always refer to the Installation and Operating Instructions of the appropriate appliance in which important information is detailed, e. g. Safety Instructions, Set-up, Electrical Connections, Disinfection, Cleaning, etc.







 Repairs above and beyond simple maintenance may only be carried out by a qualified technician or one of our service technicians.

 Prior to working on the appliance or in case of danger, disconnect it from the mains (e. g. pull the plug).


Delivery Contents

The following articles are included in the scope of delivery (possible variations due to country-specific conditions and/or import regulations):


- Wiper blade set** 2137-995-00
 – 2 x wiper blade clamping part
 – 2 x wiper blade cover

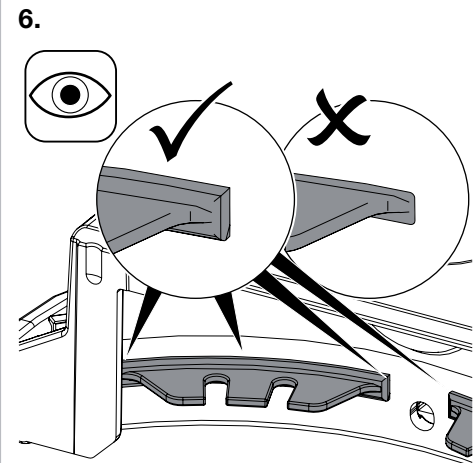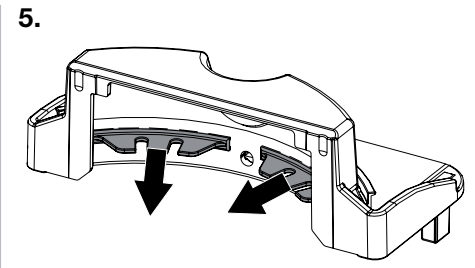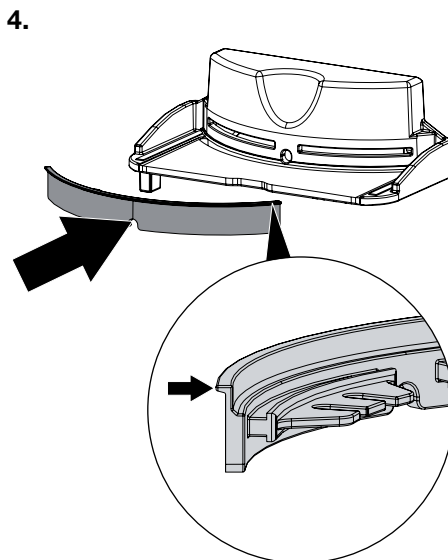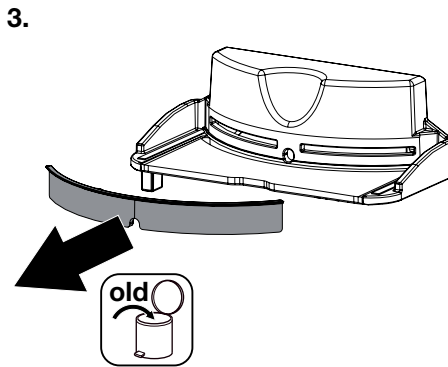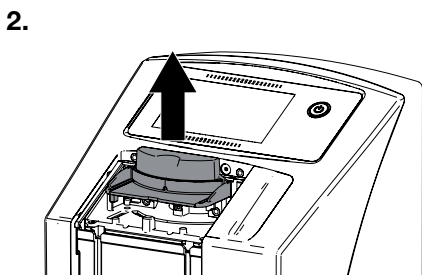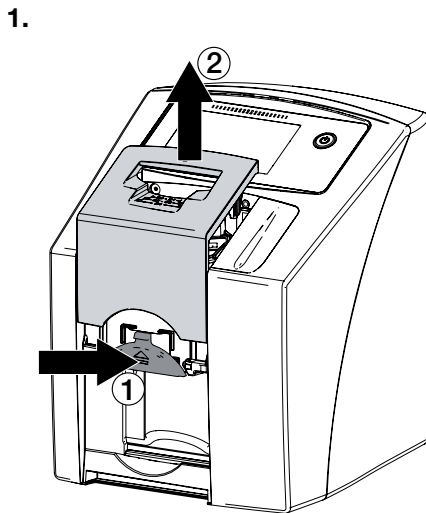


7.

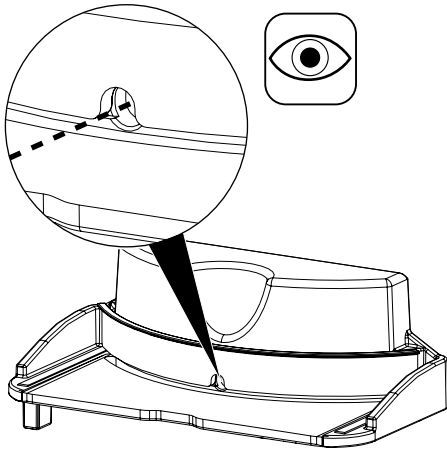


8.

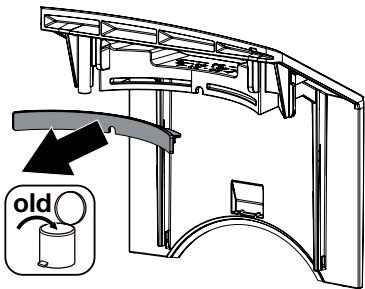


9.

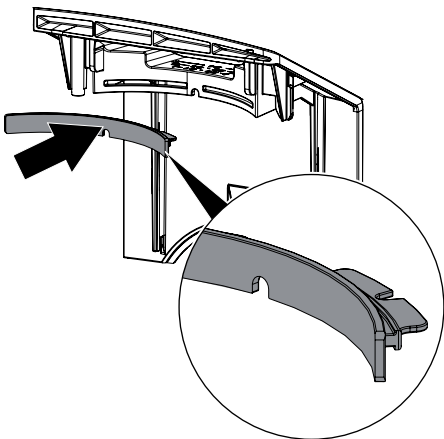

10.

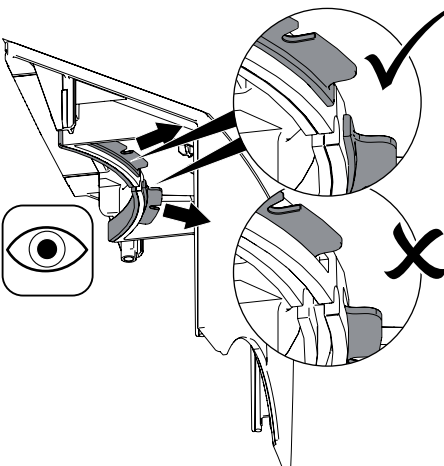

11.

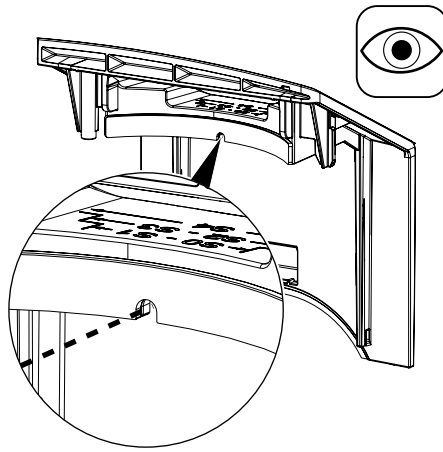

12.

- Install the appliance again in reverse order.

Tests

Monitoring the initialisation

1.

- Switch on the appliance.

2.

- Monitor the initialisation (display functions, all parts move perfectly, drives run perfectly, erasure unit illuminates).

There must be no unusual noises and nothing that is not expected from the unit should appear on the display.

Function of the components

1.

- Expose the image plate with any object (e.g. key).

2.

- Read the image plate with erasure switched on.

3.

- Delete the read X-ray image.

4.

- Homogeneously expose the same image plate and read again.

No artefacts or irregularities (e. g. ghost images, shadows) may be present.