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Repair work on the Scanner beyond normal maintenance procedures must only be carried out by Dürr NDT trained and authorized professionals or our customer service. The assembly and user instructions manual for the device must also be followed. This gives important information on safety, set-up, electrical connections, disinfection, cleaning, etc. staff.

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
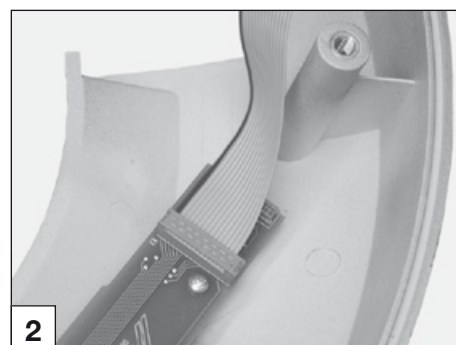
Switch off all power before opening the appliance. Verify the disconnection and assure it.

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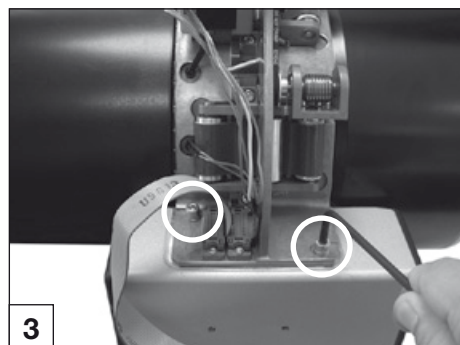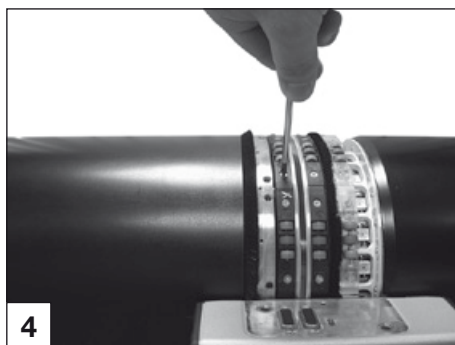
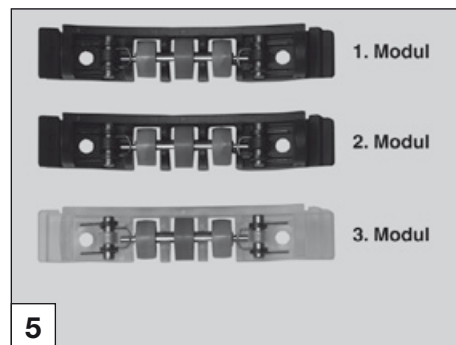
- Remove the cross-headed screws from the roller modules and replace the roller modules. Use the attached self-locking screws for fixing the new roller modules.

Make sure that no hair of the light protection brush is squeezed during mounting.

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
There are different springs with varying tensions in the roller modules.

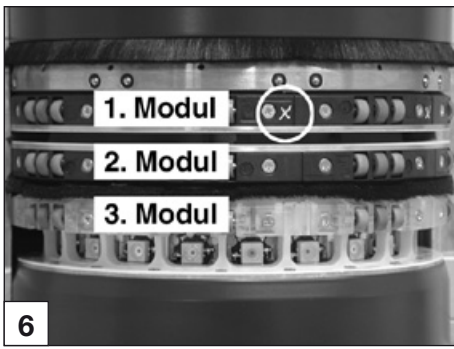- 1st roller module = silvery spring**
- 2nd roller module = red spring**
- 3rd roller module = red spring and transparent**

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
- Mount the new rollermodules.
Tighten the screws of the black rollermodules with 25 Ncm and the screws of the transparent rollermodule with 15 Ncm.

Make sure that no hair of the light protection brush is squeezed during mounting.

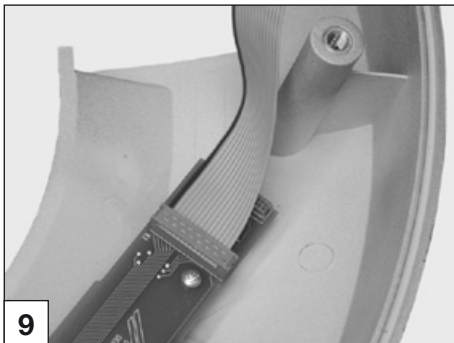**The first module is marked with an „X“. It has to be mounted under the upper light protection brush.
The 3rd module is transparent. It has to be mounted under the lower light protection brush.
Check the elasticity of the roller springs.**

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
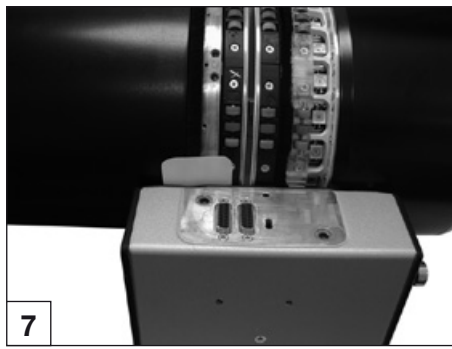
- Replace the connector to the PCB mounted on the transport arc cover.

**Assure correct positioning of connector with marker.
The red cable must face the inside of the cover.**

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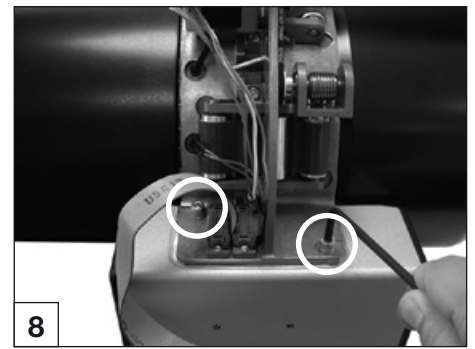- Fit the cover.

The flat cable harness must be positioned so that it does not become pinched, thereby interfering with the function of the transport arc.

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DÜRR NDT GmbH & Co. KG