

CR 35 Mobile Power Adapter

Allows the CR 35 scanner systems to be easily used in any location

The adapter can be connected to a battery pack or directly to a car cigarette lighter, any power source with 12 volts can be used, this allows maximum flexibility.



Important

This adapter is specially customized for DÜRR MEDICAL, so do not use any other adapter, this could damage the CR 35 VET. **The battery must be purchased separately and is not sold by DÜRR MEDICAL.**

Any 12V battery that meets the requirements can be used

- Capacity: 10 Ah (recommended, approx. 2 hr continuous operation)
- Voltage: 12 V
- Connection: Cigarette lighter socket (or adapter)
- For lead-acid batteries, SLA (Sealed Lead-Acid) type batteries are recommended

Technical data

- Input 12 V via cigarette lighter plug
- Converts 12 V to 24 V
- Max. current at 24 V: 5 A
- Power 120 W
- 15 A fuse included in cigarette lighter plug
- Efficiency $\eta > 80 \%$